# PROTAC®

INTEGRATING SENSES

## Protac SenSit® Nature Protac SenSit® Nature Junior

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni

|                 |    |
|-----------------|----|
| Dansk .....     | 6  |
| English.....    | 13 |
| Deutsch .....   | 20 |
| Suomi .....     | 27 |
| Français .....  | 34 |
| Nederlands..... | 41 |
| Svenska .....   | 48 |
| Español .....   | 55 |
| Italiano.....   | 62 |

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# English

|                                               |    |
|-----------------------------------------------|----|
| Marking .....                                 | 14 |
| Incidents - adverse events .....              | 14 |
| Traceability .....                            | 14 |
| Key to symbols - product label .....          | 14 |
| Safety - general precautions .....            | 14 |
| Use and use environment of the product.....   | 15 |
| Indications .....                             | 16 |
| Contra-indications .....                      | 16 |
| Accessories .....                             | 16 |
| Maintenance .....                             | 17 |
| Cleaning .....                                | 17 |
| Key to symbols - care label .....             | 17 |
| Dismantling the products before washing ..... | 18 |
| Reassembling the products after washing ..... | 18 |
| Transport and storage .....                   | 19 |
| Service life and warranty .....               | 19 |
| Disposal .....                                | 19 |

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark.

Today, Protac products are sold worldwide.

All Protac products have been developed on the basis of the theories of sensory integration, and they are supported by extensive research, documentation and case studies.

The theory of sensory integration by A. Jean Ayres forms the basis of the development of all Protac products. The theory is based on how the brain processes sensory impression and how stimulation of the muscle-joint-sense and the deep tactile sense has a calming and organising effect on the sensory system.

All Protac products contain balls, and it is the movement of the balls that provide the dynamic deep-point pressure on the body and stimulate the receptors in the deep tactile sense and the muscle-joint sense. This enhances the body awareness, alleviates sensory disorder and has a calming effect on both body and mind.

Further information can be found at [www.protac.dk](http://www.protac.dk).

## Marking

The products are CE-marked in accordance with Regulation (EU) 2017/745 on medical devices.

EU declaration of conformity can be found at [www.protac.dk](http://www.protac.dk)

## Incidents - adverse events

- Incidents related to the use of Protac SenSit® Nature must be reported to Protac.
- Adverse events related to the use of Protac SenSit® Nature must be reported to the National Competent Authority and to Protac.

## Traceability

Production month, year and batch number can be found on the product label.

Do not remove the product label from the product.

## Key to symbols - product label

|                                                                                   |                                                                          |                                                                                   |                       |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------|
|  | Instructions for use                                                     |  | Maximum user weight   |
|  | Medical device                                                           |  | Article number        |
|  | Lot number                                                               |  | For indoor use        |
|  | Manufacturer                                                             |  | Production month/year |
|  | CE marked in accordance with Regulation (EU) 2017/745 on medical devices |                                                                                   |                       |
|  | Warning: See under Safety                                                |                                                                                   |                       |

## Safety - general precautions

|                                                                                     |                                                                                                                                             |
|-------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------|
|    | Do not use unauthorized accessories such as belts or harnesses in the chair. Only original Protac SenSit® Frame can be used with the chair. |
|    | Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.                                                            |
|  | Never leave switched on electrical appliances that emit heat on or near the product.                                                        |
|  | Do not use near open fire.                                                                                                                  |
|  | Do not use other balls in the product than those supplied by the manufacturer.                                                              |
|  | Incontinence cover must be kept away from children.                                                                                         |
|  | Protac SenSit® Nature does not tolerate temperatures below 0° C.                                                                            |
|  | Never leave a person in the chair when it is placed directly in the sun.                                                                    |

## Use and use environment of the product

Protac SenSit® Nature is supplied ready for use. It is for both indoor and outdoor use in both private homes and institutions.



The optimum use of the chair is with the application of both neck wings and side wings and with both feet on the puff.

Light pressure can be applied on the shoulders to increase sensory stimulation.

EN

The chair can be used with neck wings and/or side wings, depending on the user needs.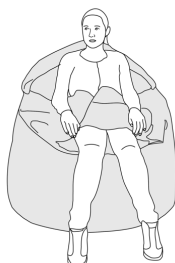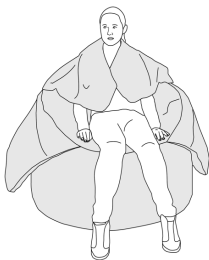

Protac SenSit® Nature Puff can be used separately as an extra seat.

During transport, Protac SenSit® Nature might take shape from the box. As soon as the chair is put to use, it will go back to its round shape again.

## Indications

Protac SenSit® Nature is a sensory-stimulating aid intended to compensate for a permanent disability. The ball chair provides an enveloping feeling of the whole body. It enhances body awareness, alleviates symptoms of sensory impairment and has a calming effect on body and mind. It also alleviates physical and mental unrest and sharpens concentration.

EN

The junior size chair has been developed for children aged 3-10 years.

The shapeable content of sensory-stimulating balls in seat, back and wings provides a safe and enveloping feeling for the user. The neck cushion has granulate filling for optimum head support.

The ball chair is for use during the day and can be used to regulate arousal.

## Contra-indications

Persons with pressure ulcers or persons at risk of developing pressure ulcers are not to use the product.

Persons with a history of chronic attacks or neurological disorders are not to use the chair unattended.

Persons unable to take care of themselves are not to use the chair unattended. Special attention must be given to users unable to express discomfort, e.g. they might not like being covered by the chair wings.

Each chair model has a maximum user weight limit. See page 3.

## Accessories

A carry bag is available, see page 5:

- To be used to protect the chair when it is placed in storage.
- Makes transporting the chair, e.g. in a car, easier.

## Maintenance

Over time a recess will form in the seat as the granulate at the bottom 'settles' through use. It is therefore important to turn the chair upside down once in a while and shake it a little to allow the granulate to become evenly distributed again. This will restore the shape of the seat.

After heavy rain, turn the chair upside down and wipe off the bottom with a dry cloth.

If the chair is placed directly in the sun, it is recommended to protect it with the rain cover when not in use, to protect it against fading in the sun. The fabric will eventually fade, but the rain cover will postpone the process.

Once a year, it is important to check the seams and the general condition of the chair. If repair is needed, this must be done before using the chair.

## Cleaning

Protac SenSit® Nature er hygienic, water-repellent and has taped seams.

Chair and puff can be wiped off with a cloth wrung out in warm soapy water. Both chair and puff can be cleaned with anti-septic wipes.

The frequency of washing depends on use.

The cover and the ball bags can be washed in a washing machine in accordance with the care label.

The products can be dismantled and reassembled by following the procedure described on the next page.

Protac does not recommend the use of stain remover.

## Key to symbols - care label

|                                                                                   |                             |                                                                                   |                       |
|-----------------------------------------------------------------------------------|-----------------------------|-----------------------------------------------------------------------------------|-----------------------|
|  | Maximum washing temperature |  | Line dry in the shade |
|  | Do not wash                 |  | Do not iron           |
|  | Can be tumble dried         |  | Do not dryclean       |
|  | Do not tumble dry           |  | Do not bleach         |

A video showing how to assemble the chair is available at:  
<https://protac.dk/products/protac-sensit-uk>

## Dismantling the products before washing

Chair:

1. Remove granulate innerbag from neck cushion. NB! Do not wash the innerbag!
2. Remove ball bags from side and neck wings. Ball bags to be washed separately in washing bag (enclosed) in accordance with the care label.
3. Unzip the seat cover - must be zipped back together with the rest of the cover before washing.
4. Release the large inner sack from velcro (back and bottom) and take it out.  
NB! Do not wash the inner sack!
5. Incontinence cover can be removed and washed separately in accordance with the care label.
6. Unzip the bottom - must be zipped back together with the rest of the cover before washing.
7. Turn the cover upside down and unzip the chambers that contain front and back plates.
8. Remove front and back plates - NB! Do not wash the plates.
9. Close all zippers attaching the seat cover and the bottom to the rest of the cover before washing it in accordance with the care label.

Puff:

1. Remove the ball/granulate inner bag from the cover. NB! Do not wash the inner bag.
2. Close the zipper on the cover before washing it in accordance with the care label.

## Reassembling the products after washing

Chair:



1. Unzip the seat cover and the bottom from the rest of the cover and turn the cover upside down.
2. Unzip the chamber that holds the back plate and place the back plate inside.  
NB! Make sure that hard plastic faces inwards.
3. Close the back plate chamber.
4. Unzip the chamber that holds the front plate and place the front plate inside.  
NB! Make sure that hard plastic faces inwards.
5. Close the front plate chamber.
6. Reattach the bottom of the cover by closing the zipper. Turn the chair around.
7. Reattach the incontinence cover onto the inner sack.
8. Place the inner sack inside the cover. Make sure the velcro pieces fit at the back.
9. Reattach the seat cover. Make sure the velcro piece fits on the inner sack.
10. Place the ball bags for side wings and neck wings in the wings. Make sure the velcro pieces are attached at the top inside each wing.
11. Place the granulate innerbag in the neck cushion.
12. Close the zipper from side wing to side wing.

Puff:

1. Open the cover and place the ball/granulate innerbag inside.
2. Close the zipper.

## Transport and storage

Always use the handles on the side and on the back of the chair when moving it. Lifting the chair is a 2 person job.

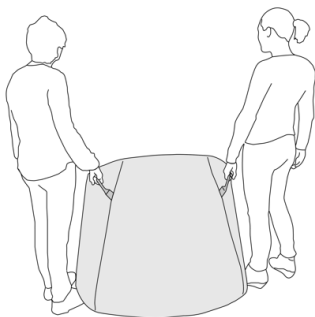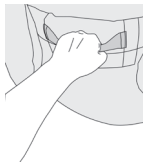EN

Always ensure using the handles when pushing or pulling the chair even over short distances. Note that this will cause wear on the bottom, and it is always recommended to lift the chair.

## Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

## Disposal

The product can be broken down into the following components for disposal:

- The plastic balls can be disposed of as hard plastic.
- The granulate can be disposed of as combustible waste.
- The fabric can be disposed of as textiles.

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Protac A/S