Protac Kugledynen™

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GB - Directions for use

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SE - Bruksanvisning

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Protac Kugledynen™ Classic

Protac Kugledynen™ Flexible

Protac Kugledynen™ Calm

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Protac Ball Blanket™ Flexible

Protac Ball Blanket™ Calm

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts. Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe and Australia since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac's plastic balls are TÜV-approved and the cotton and flame-retardant textiles are OEKO-certified.

For more details, go to www.protac.dk

GB

Labelling

Our products are labelled according to legislation concerning medical equipment.

The label with details of manufacturer, raw materials and washing instructions is sewn into the foot of the blanket to avoid irritating the user. This location also protects the label from wear and tear.

Key to label symbols: please refer to **Key to symbols**

General precautions

- ⚠ • **Safety**
- ⚠ • **Not for unsupervised use** by users unable to care for themselves.
- ⚠ • **Not for unsupervised use** by users with low lung capacity.
- ⚠ • Always carefully monitor a user's reaction. There can be indications of over-stimulation if the user does not like lying under the blanket. This can be seen in the form of the user sleeping too long, or becoming hyperactive.
- ⚠ • When the ball blanket is used with an incontinence cover, it is important to be aware of the possible risks.
The same risk precautions apply as for the use of a plastic bag:
 - A ball blanket used with an incontinence cover **must not** be used for children with out supervision.
 - A ball blanket used with an incontinence cover **must not** be used for patients unable to care for themselves **without supervision**.
 - A ball blanket used with an incontinence cover **must not** be pulled over the head.
- ⚠ • Ball blankets with polystyrene/polypropylene balls contain small particles, and **must not** be used for children below the age of 3 **without supervision**.
- ⚠ • Smoking is forbidden under the blanket. Embers from cigarettes can cause burns.
- ⚠ • Never leave a computer or other electrical appliances switched on in a bed with a ball blanket, as they emit heat which can be harmful to the blanket, or in extreme circumstances, cause the appliance to ignite, due to a build-up of heat.

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac Ball Blanket was developed to help sufferers of function impairment.

Children, young and old people can benefit from the Protac Ball Blanket:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

GB

The Protac Ball Blanket can be used for patients with brain damage resulting in spasticity. The weight and pressure of the balls help suppress spasticity, and calm the body of patients with brain damage. Furthermore, the patient benefits from enhanced body awareness.

The Protac Ball Blanket is used by ADHD and autism sufferers to sleep with, and as a source of comfort during the daytime. The pressure and weight of the balls impart sensory stimuli which help define the body, giving comfort and calm. ADHD sufferers often have a problem winding down at night, and the Ball Blanket gives the body a signal to relax through its pressure and weight.

Psychiatric patients often suffer from muscular and psychological unrest. The Protac Ball Blanket is used to create calm and comfort for the user. The weight and pressure of the balls also give a better feeling of body awareness and enhanced wellness. The blanket is used to sleep with, and to provide comfort during the daytime.

The Protac Ball Blanket is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

The blanket is used for sufferers of stress to provide calmness, comfort and better body awareness. It is used to sleep with, and to provide comfort during the daytime. Sufferers of stress often have a problem feeling their body, preventing them falling asleep at night. The pressure and weight of the blanket give the body a signal to relax.

The Protac Ball Blanket can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. The blanket can be used in schools in classrooms or association areas, for instance. At home, the blanket can be used in a bedroom or the lounge.

Using the product

The Protac Ball Blanket is supplied with a dust cover. An ordinary duvet cover should be used over this. **The dust cover shall be removed if the customer keeps the blanket.**

The Protac Ball Blanket is used in the same way as an ordinary duvet for sleeping. Pack it tightly around the body so that you can clearly feel the balls.

We recommend sitting with the duvet over you on a chair or sofa initially, to feel the weight of the balls, which will be a new experience.

For extra simulation, place your hands on the blanket and press the balls against your body.

You can also lie on top of the blanket. This will provide sensory stimulation to your back.

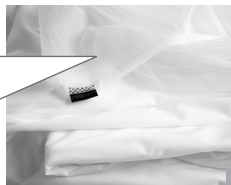
If the blanket is to be used by a child, we recommend allowing the child to play with it first while awake by crawling under it, or sitting with the blanket over his/her legs.

Accessories

Protac Incontinence Cover

Using the product

The Protac Incontinence Cover is used over the outside of the Ball Blanket if the patient is incontinent (involuntary urination). An ordinary duvet cover should be used over the incontinence cover for comfort (see the Protac Incontinence Cover directions for use).

GB

General precautions

- ⚠ • We recommend that the opening of the incontinence cover is at the opposite end to that of the duvet cover (not applicable for incontinence cover with mesh top)
- ⚠ • The same risk precautions apply as for the use of a plastic bag:
 - A ball blanket used with an incontinence cover **must not** be used for children without supervision.
 - A ball blanket with incontinence cover **must not** be used for patients unable to care for themselves without supervision.
 - A ball blanket used with an incontinence cover **must not** be pulled over the head.
- ⚠ • We recommend using the incontinence cover version with a mesh top (breathable material) for users unable to care for themselves.

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

All Protac Ball Blankets can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac Ball Blanket, Classic

- Ball blanket, item nos. 100-101, 100-102, 100-105, 100-106, 100-109, 100,110 can be machine washed in a min. 16 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the blanket.
- Ball blanket, item nos. 100-119 with polypropylene balls can be split in two and washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the cover.
- Ball blanket, item nos. 100-107 baby can be washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the cover.

Protac Ball Blanket Flexible

- Ball blanket, item nos. 100-201, 100-205, 100-202, 100-206 can be machine washed in a min. 6 kg washing machine with washing powder with no bleaching agents. Remove the bags from the cover. Wash cover and bags separately - see also the washing instructions on the blanket.

Protac Ball Blanket Calm

- Ball blanket, item nos. 100-301 and 100-302 can be split in two and washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the blanket.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring.

GB Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend hanging the blanket out to air on a regular basis.

Transport and storage

The Protac Ball Blanket is supplied in a handy carrying case.

The blanket is ready for use.

When transporting, pack the blanket in its case and a sheet of plastic to protect it from damp and dirt.

If you are not using the blanket, store it carefully in its case to protect from damp and dirt.

Disposal

Protac Ball Blanket can be broken down into the following components for disposal: Plastic and textile.

All components can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- overloading
- misuse
- non-maintenance or incorrect maintenance

Traceability: The month and year of manufacturer are marked on the washing instructions label.

Symbol explanation

Washing bowl: Maximum washing temperature 60°C

Warning, see relevant documents

Triangle:
Do not bleach

User information

Square with a circle and dot:
umble drying possible,
low temperature max. 60°C

For indoor use

Iron:
Do not iron

CE approval according to Directive concerning medical devices 93/42/EC

Circle:
Do not dry clean

Manufacturer

Product serial number

GB

List of spare parts

Diagram – technical specifications

Dimensions:

Protac Ball Blanket, Adult

length: 200/220 cm.

width: 140 cm

Protac Ball Blanket, Junior:

length: 140 cm.

width: 100 cm

Protac Ball Blanket, Baby:

length: 100 cm.

width: 70 cm

Protac Ball Blanket, Premature:

length: 50 cm.

width: 50 cm

Patent: Protac Ball Blanket Calm is registered as a Utility Model

Protac Ball Blanket Flexible is registered as a Utility Model

CE-approval: The Protac Ball Blanket has CE-approval according to: EN 12182:2012

Fire test: BS 7175: 1989 (2008)

Ignition Sources 0 and 1

BS 5852: 1990 BS EN 1021 1/2

Service information

No other balls can be used in a Protac Ball Blanket than those used by the manufacturer.

Guide

- to daily inspection of the Protact Ball Blanket before use

No.	Check	Remarks
1	<ul style="list-style-type: none">Blanket appears intact	
2	<ul style="list-style-type: none">No sign of holes, damage to the textile or seams	

GB

Protac A/S

Kystvejen 17, 1.

DK- 8000 Aarhus C

Tel.: 86 19 41 03

e-mail: protac@protac.dk

Dealer list:

Please refer to: www.protac.dk/kontakt

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