Protac SenSit® & Protac SenSit® Puff

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GB - Directions for use

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SE - Bruksanvisning

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Protac SenSit®, standard

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Protac SenSit® Puff

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Protac SenSit®, tall

Protac SenSit® Puff

You have chosen a Protac quality product. Protac guarantees that you have purchased an original Danish design product developed in partnership with professionals.

Since 1994, Protac has developed, manufactured and sold sensory stimulating aids to the healthcare sectors in Denmark, Europe and Australia.

Protac's products are developed from the theory of sensory integration and are well documented with respect to research, documentation and case descriptions.

Protac's production process focuses on environment and safety, and Protac's plastic balls are TÜV approved and both polyester and flame-retardant fabrics are eco-certified. For more information, go to www.protac.dk – includes instructional video on how to change the cover.

GB

Labelling

The products are labelled in accordance with medical device laws.

The label is placed inside the chair to prevent discomfort when sitting in the chair and to protect the label from wear.

A label with information about the manufacturer, raw materials and washing instructions is fitted inside the individual parts of the chair and is found by unzipping the covers of the following parts:

- chair cover
- seat cover
- inner bag with balls
- inner bags in neck and side wings
- inner bag in neck pillow
- foam in back
- foam in front edge

Symbol explanation on label: see the **Symbol explanation** section

General safety rules

- ⚠ Please be aware of how the user reacts when sitting in the Protac SenSit chair. If the user does not like being embraced by the chair's neck and side wings, it may be a sign of overstimulation, anger or anxiety.
- ⚠ Make sure that the user does not overheat if he/she is embraced by the chair's neck and side wings.
- ⚠ Users with poor breathing must use the side and neck wings with care and **never unsupervised**.
- ⚠ The chair **must not** be used **unsupervised** with individuals who require special care.
- ⚠ Check to see whether any balls might be irritating the user. This is particularly important if the user is incapable of independent movement.
- ⚠ The chair **must not** be used by individuals with bedsores.
- ⚠ Make sure that children and other users do not get their hands and feet caught in the carrying straps.

- ⚠ • The Protac SenSit chair is not designed for outdoor use.
- ⚠ • The Protac SenSit chair may only be used by individuals with a maximum weight of 200 kg.
- ⚠ • Avoid smoking in the chair. Lost embers or cigarettes can cause burns.
- ⚠ • Never leave a turned on computer or other electrical devices in the chair as they generate heat which may cause the cover to deteriorate, or in the worst case cause the electrical unit to ignite as the heat cannot escape.

Purpose and function of the product

All Protac's products are developed from the theory of sensory integration. The theory originates in the USA and was developed by occupational therapist and psychologist A. Jean Ayres back in the 1970s. The theory is based on how the brain processes sensory impressions and how the different sensory stimuli can be influenced through weight and pressure on the tactile sense (sense of touch) in the skin and the proprioceptive sense (sense of movement) in muscles and joints. The pressure, movement and weight of the balls stimulate these particular senses. The Protac SenSit chair was developed as an aid for individuals with functional impairments.

GB

The Protac Sensit chair is suitable for children, youths, adults and the elderly.

- Brain damage, including spasticity, Huntington's chorea, Parkinson's disease etc.
- ADHD and autism
- Mental illnesses such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Developmental disorders, sensory disruptions, deaf-blindness etc.
- Stress

The Protac SenSit chair is used to give relief or as a supplement for individuals with brain damage who are wheelchair users or bedridden. The chair's stable shape supports the neck, head and body, and the balls in the seat and back make it easy to adjust the chair and create a comfortable sitting and resting position.

Individuals with ADHD and autism can use the Protac SenSit chair as a 'contained space' where the user is surrounded by the balls and receive the sensory stimuli needed to ensure calm and a sense of security. The chair can be used to eliminate sensory impressions so that the user can concentrate on school homework for example.

Individuals with mental illness often suffer from motor and mental restlessness. The Protac SenSit chair can be used as a 'safe place' to ensure calm and a sense of security for the user. The weight and pressure of the balls also leads to better body awareness and improved well-being.

The Protac SenSit chair is used for individuals with for example developmental disorders, sensory disruption and deaf-blindness in order to increase the body's sensory stimuli.

The Protac SenSit chair is used for individuals with stress to ensure calm and a sense of security as well as better body awareness.

The Protac SenSit chair can be used in institutions, care facilities and hospitals, for example in common rooms or private rooms. In schools the Protac SenSit chair can be used in the class room, in the library or in common rooms. In private homes the Protac SenSit chair can be used in the user's room or in the living room.

Using the product

The Protac SenSit chair can be used with the neck wings placed inside the chair, behind the user's back and with the side wings hanging down the side of the chair. This contains the body, without the user feeling the weight of the wings, which may be a good start.

The Protac SenSit chair can be used with the neck and/or side wings placed over the user so he/she can feel the weight and pressure of the balls. This leads to a better sense of the body's boundaries and body awareness.

When the user is wearing the neck wings, the care staff/ helper may apply light pressure to the shoulder area to increase the sensory stimuli in the shoulders.

The Protac SenSit chair can be moved by using the carrying straps.

When using the Protac SenSit chair for individuals with for example spasticity, create a recess at the back of the seat by moving the balls so that the user can sit in a reclined position further back in the seat.

The Protac SenSit chair is delivered fully assembled

Dismantling instructions before wash:

1. Take the neck pillow out of the neck compartment.
2. Seat zips off.
3. Loosen the inner shell with balls using the back Velcro.
4. Unzip the components with front and back foam plates.
5. Empty the balls from both neck wings (two bags).
6. Remove the inner shell.
7. Remove the back foam plate.
8. Remove the front foam plate.
9. Empty the balls from both side wings (two bags). The balls are washable, but as they take up a lot of space, it is advisable to remove them.
10. All zippers should be zipped up before the chair cover is washed.

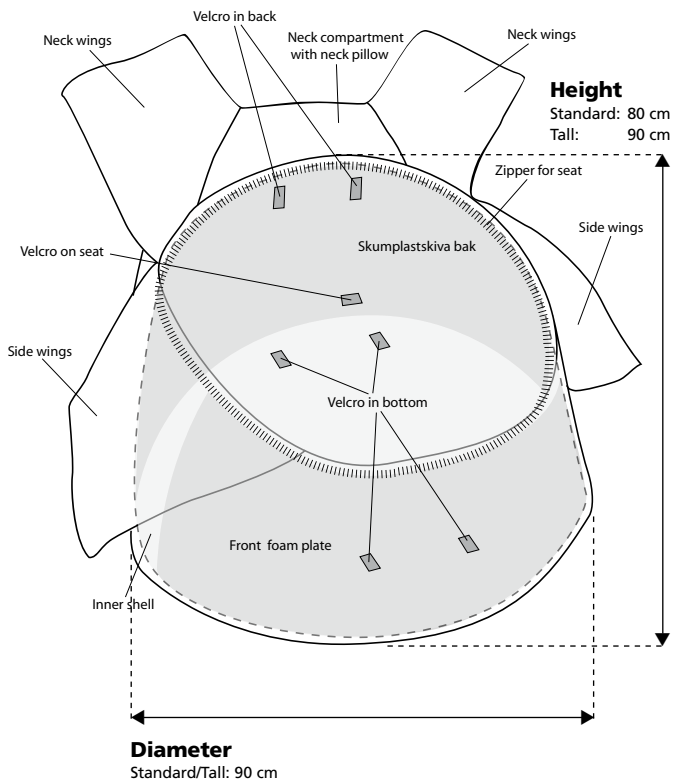
GB

Assembly after wash:

1. Unzip the seat cover.
2. Spread out the chair cover so Velcro on the inside of the bottom of the cover is visible.
3. Unzip the front foam cushion compartment.
4. Place the foam cushion in the compartment. Fold in both sides of the cushion when fitting – it can be smoothed out again once it is in place.
5. Zip the compartment.
6. Unzip the compartment for the rear foam cushion.
7. Place the foam cushion in the compartment – remember the hard plastic side must face inwards. Again, fold both sides of the foam cushion to ease placement.
8. Zip the compartment.
9. Position the inner bag in the cover so that the Velcro tapes meet at bottom and back.
10. Zip the seat cover. Remember to affix the Velcro on seat cover to the Velcro on the inner bag (in the centre, close to the rear bag).
11. Place the neck cushion and neck wings into the compartments and zip. Remember to secure the neck wings with Velcro.
12. Place side wings (the 2 large bags with plastic balls) into the cover and zip.

Find videofilm showing how to assemble the SenSit chair at: www.protac.dk

Please note! This is a job for two people!

Accessories

Protac SenSit Puff

The Protac SenSit Puff can be used as a foot/leg rest, together with the Protac SenSit chair.

The Protac SenSit Puff can be used as a stool.

GB

General safety rules

- ⚠ • Must not be used by individuals with pressure ulcers on their calves and heels.
- ⚠ • Take care not to trip over the footstool if the user is unsure of his/her balance.

Product lifetime

The product's expected lifetime is five years.

The product's lifetime is based on correct use, cleaning, washing, service and maintenance. Products that are used in excess of the above lifetime must be assessed for future use. The user is responsible for continuously assessing whether the product can be used in excess of the lifetime stated.

Cleaning

It is recommended to clean and wash the covers as the Protac SenSit chair may become dirty over time. The need and frequency of washing depend on the use of the chair. Follow the instructions below when washing and cleaning the chair.

- The outer fabric can be washed in a washing machine, min. 5 kg, using a washing powder without bleach. See also the washing instructions inside the cover.
- The seat cover with an incontinence cover on the reverse side can be unzipped and washed separately using a washing powder without bleach. See also the washing instructions inside the cover.
- Neck and side wings can be washed in a washing machine, min. 5 kg, using the supplied washing bags and a washing powder without bleach. See also the washing instructions inside the wings.

Before using any stain removers, do a test on a non-visible area (bottom of chair) to make sure that it does not damage or change the colour of the fabric. Protac does not recommend the use of stain removers.

Protac accepts no liability for damages resulting from incorrect washing and maintenance.

- ⚠ • Do not wash the inner bag with balls
- ⚠ • Do not wash the inner bag in the neck rest

Maintenance

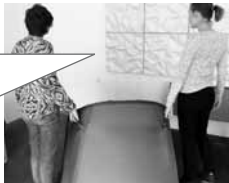
Over time a recess will form in the seat as the small balls at the bottom 'settle' through use. It is therefore important to turn the chair upside down once in a while and shake it a little to allow the balls to become evenly distributed again. This will restore the shape of the seat. Inspect the chair for defects and deficiencies prior to each use, and inspect the chair's seams, zips, fabric, straps, balls and general condition once a year. If serious defects or deficiencies are discovered, the Protac SenSit chair must not be used until the defects have been repaired and safety has been restored.

GB

Transport and storage

It is easy to move the Protac SenSit chair using the two carrying straps fitted on each side of the chair.

- ⚠ The chair should preferably be lifted by two people using one strap each.

- ⚠ When moving the chair, do not lift but pull on the straps as this puts less strain on the back. However, if moving the chair over a distance of more than two metres, the chair should be lifted to prevent unnecessary wear to the fabric.

The Protac SenSit chair is ready to use on delivery.

During transport, the chair is wrapped in protective plastic in a cardboard box, which protects the chair against dirt and moisture.

The chair must be stored in a safe place when not in use, preferably covered to protect it against dirt and moisture.

Disposal

The Protac SenSit chair can be divided into the following components for waste disposal purposes: Plastic and fabric.

All parts of the chair can be disposed of as flammable waste.

Waste must be disposed of in accordance with applicable rules.

Warranty

There is a two-year warranty against manufacturer's defects.

The warranty will be voided in the following cases:

- Overload of the Protac SenSit chair
- Harsh use of the Protac SenSit chair
- Lack of or incorrect maintenance of the Protac SenSit chair

Traceability: Production months and year are marked on the backside of the washing label.

Symbol explanation

Washing bowl: Maximum washing temperature 60°C

Warning, see relevant documents

GB

Triangle:
Do not bleach

User information

Square with a circle and dot:
Tumble drying possible,
low temperature max. 60°C

For indoor use

CE approval according to Directive concerning medical devices 93/42/EC

Iron:
Do not iron

Manufacturer

Circle:
Do not dry clean

Product serial number

List of spare parts

- Banana-shaped cushion with polypropylene balls
- Seat cover
- Foam for back
- Foam for front edge

Scale drawing – technical specifications

Dimensions:

Protac SenSit chair, standard Height: 80 cm. diameter: 90 cm. weight: approx. 18 kg.

Protac SenSit chair, tall Height: 90 cm. diameter: 90 cm. weight: approx. 18 kg.
(see drawing)

Patent: The Protac SenSit chair is patent pending.

CE marking: The Protac SenSit chair is CE marked according to EN 12182:2012.

Crash test: EN15377: 2007

Fire test, UK: BS 5852: 3006 Clause 11, ignition source 5

Service information

Only the balls used by the manufacturer may be used in the Protac SenSit chair.

Guide

- to daily pre-use inspection of the Protac SenSit chair

| No. | Check | Note |
|-----|---|------|
| 1 | <ul style="list-style-type: none">All parts are in place, no loose parts. (wings and neck pillow) | |
| 2 | <ul style="list-style-type: none">Straps are not loose, seams are OK. | |
| 3 | <ul style="list-style-type: none">Straps are not loose, seams are OK. | |
| 4 | <ul style="list-style-type: none">Seat height is OK (no sinking of the seat). | |
| 5 | <ul style="list-style-type: none">No sign of holes or damage to the fabric and seams | |

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Protac A/S