Protac KneedMe®

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GB - Directions for use

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SE - Bruksanvisning

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Protac KneedMe®

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GB - Directions for use

Protac KneedMe®

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts.

Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe and Australia since 1994.

Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies.

Protac focuses strongly on protection of the environment and safety during its production process. Protac's plastic balls are TÜV-approved and the polyester and flame-retardant textiles are OEKO-certified.

GB

For more details, go to www.protac.dk

Labelling

Our products are labelled according to legislation concerning medical equipment.

The label with details of manufacturer, raw materials and washing instructions is sewn inside the cover to avoid irritating the user. This location also protects the label from wear and tear.

Key to label symbols: please refer to **Key to symbols**

General precautions

Safety

Please note that the Protac KneedMe is not intended as a restriction of personal freedom or as a safety device.

- ⚠ • The knee rug's belt should only be fastened around the user's waist, **never** around a chair back. The belt prevents the rug sliding downwards during use. When correctly adjusted and tightened, the belt should fit closely around the user - comfortably but not tight.
- ⚠ • Users unable to open the belt's Velcro fastening themselves and remove the rug without help should always be kept under supervision.
- ⚠ • If a user's movements or impaired cognitive abilities accidentally cause the rug to unfasten, we recommend keeping the user under supervision.
- ⚠ • Take precautions to avoid pressure sores when using heavy products.
- ⚠ • Cease using the product if there are any signs of redness or irritation on the user's thighs or buttocks as a result of using the rug.
- ⚠ • **Not suitable for use** on children.

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac KneedMe was developed to help sufferers of function impairment. It can be used by adults and the elderly.

Protac KneedMe is used for patients suffering from physical and psychiatric restlessness. Sudden, violent movements and aggressive behaviour are common, which recede when wearing the rug. Patients become more approachable when in contact with others. The Protac KneedMe can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. At home, the rug can be used in a bedroom or the lounge.

Using the product

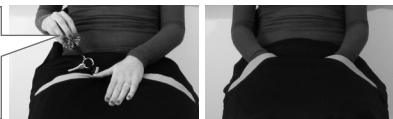
GB

The Protac KneedMe is laid over the user's lap.

If it slips down during use, the straps can be tightened around the waist (the straps are accessible by opening the zip).

A carabiner hook is sewn into the top of the pocket for objects such as keys which the user likes to touch.

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

Protac KneedMe rugs can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac KneedMe

- can be machine washed in a min. 6 kg capacity washing machine with washing powder with no bleaching agents - see also the washing instructions on the cover.
- The cover and ball bag can be washed separately. The laundry bag supplied is recommended for washing.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend airing the rug regularly.

Check the Protac KneedMe every time it is used for defects and deficiencies, and inspect the seams, zip, fabric, balls and general condition once a year. If any serious defects are found, withdraw the rug from use until repairs have been made and it is safe to use again.

Transport and storage

The Protac KneedMe is supplied in a handy carrying case.

The rug is ready for use.

When transporting, pack the rug in its case and a sheet of plastic to protect it from dirt and damp.

If the rug is not in use, store it carefully in its bag to protect from dirt and damp.

Disposal

The Protac KneedMe rug can be broken down into the following components for disposal:

Plastic and textile.

All components can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

GB

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

Traceability: The month and year of manufacturer are marked on the washing instructions label.

Key to symbols

Washing bowl: Maximum washing temperature 40°C

Warning, see relevant documents

Triangle:
Do not bleach

User information

Square with a circle and dot:
umble drying possible,
low temperature max. 60°C

For indoor use

CE approval according to Directive concerning medical devices 93/42/EC

Iron:
Do not iron

Manufacturer

Circle:
Do not dry clean

Product serial number

CE-approval: Protac KneedMe is CE-approved according to EN 12182:2012

Service information

No other balls can be used in a Protac KneedMe rug than those used by the manufacturer.

Guide

- to daily inspection of the Protact KneedMe before use

No.	Check	Remarks
1	<ul style="list-style-type: none">• Rug appears intact	
2	<ul style="list-style-type: none">• No sign of holes, damage to the textile or seams	

GB

Protac A/S

Kystvejen 17, 1.

DK- 8000 Aarhus C

Tel.: 86 19 41 03

e-mail: protac@protac.dk

Dealer list:

Please refer to: www.protac.dk/kontakt

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