## Vedrørende miljøbeskyttelse:

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## UK:

Koko-Nora® the Big walrus Pilo is a heavy blanket for children, and people with special needs, who may have difficulty in falling asleep or just need a time out during the day. Let the body of Pilo lay on top of the child's stomach/chest and let the feet give a light pressure to the body/ shoulders. What happens in a body when it is exposed to stimulation, as pressure, is that it releases Oxytozin, which is de-stressing and gives a calm feeling to a stressed body. Pilo can also be placed on the person's lap and have the feet placed on each side of the hips, which also gives a light pressure and have an effect on the sensoric stimulation of the body. If Pilo will be used to hang on the shoulders, the person may not collapse in the back, since this may cause damage to the back of the person, in case this occur Pilo must be removed immediately. Pilo has a soft surface, which also gives tactile stimulation to restless fingers. Who knows maybe Pilo can give less sleepless nights.... Just pay attention to one thing, the person have to be able to remove it by themselves, in case they want to sleep without.

The Koko-Nora® product is designed with love, to give calmness and safety, for children with an unrestfull body from the age +3 years, and to people in general with special needs during the day.

Size: H-30mm W-540mm L-680mm weight: 2,7kg

Composition: 100% Polyester  
Composition inside: 97% Polyethylene  
3% Polyester

Koko-Nora® only uses qualities that do not contain carcinogens. Therefore, no flame retardants have been used on the product.

### Other practical informations:

Care instructions are at the back of the product. Koko-Nora® recommend that superficial dirt will be removed with a damp cloth and soap, since repeatedly wash will wear on the product and the warranty does not cover this. To protect the material, only use the product indoors. Keep the product away from sharp and hot objects as well as open flames. Koko-Nora® would like to guarantee that our products can withstand all kinds of bites, but we cannot. Teeth are counted as very sharp objects. If the person bites in the product, the product may leak, and therefore you should always inspect the product before use. It is recommended to use both hands when moving the product. Do not pull the product in any way. If you are transporting the product in a vehicle, the product must be secured to prevent it from flying around and injure anyone. If there is a hole in the product, the product must be removed from the person immediately. The Koko-Nora® product should be checked regularly for damage, as loose objects and its contents can be dangerous for children less than +3 years of age as well as people with dementia. Do not put the product in the microwave or another heat source.

### Regarding environmental protection:

Help the environment by not disposing your product in the household waste. Cut the product into pieces and distribute the individual components such as plastic and fabric and according to general local regulations – in the relevant recycling systems.

### Important:

Make sure that when the product is in use that it is used correct follow the instructions given. It is important to keep an eye on the person during use as the product is heavy and that can affect the body physically. The Koko-Nora® product should only be used with the person's consent. If the person feels discomfort when using the product, and rejects the product on several occasions, the product should be removed and not used for this person. If the product is accepted, the product should be seen as an assistive device against restlessness and stress and used in limited time intervals depending on the person's needs. When starting the use of this product, start with 5-10 min and see what effect it gives and then increase the time interval as needed. Be aware that the person does not get too used to the product, so the effect wears off. The person may though sleep with the product all night if this is wanted. Once the desired effect is achieved, the product must be removed with care. Do not throw with the product, as the weight in it can cause damage both to the product but also to people in the vicinity of the product.

### Warnings:

If the person is severely restricted by motoric and/or physical conditions, the product should be used with special caution. Do not use the product if the person cannot remove it by themselves! If the product is hanging on the persons shoulder, then it is important that the person do not collapse in their back during use. If this occur then remove the product immediately, since the weight may harm the back of the person. For safety reasons, we generally recommend that people using an Koko-Nora® product should be under supervision during use. After the use of the product, the persons who require supervision must be kept out of reach of the product. Keep the airway clear when using the product. If the person has difficulty breathing, do not place the product on the chest, as the weight can have a negative effect on breathing and cause a choking hazard. All products with weight should be used with caution, as weight i.a. can cause blockage of blood circulation. An Koko-Nora® product should not be used near water as it may cause drowning.

### Warranty:

In the event of manufacturing defects, Koko-Nora® provides a two-year warranty. This warranty is only valid if the product is used as directed. Damage caused by incorrect or improper use excludes the use of the warranty. The product can be transported in a bag. When used correctly, you get a product that lasts a long time.

For further questions, please feel free to contact Majbritt Hillerup. Keep this original instruction manual for future reference.

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Oberstoff: 100% Polyester  
Futter: 100% Polyethylen  
3% Polyester

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