Snoezle®

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User manual - Manuel d'utilisation - Brukerhåndbok

Versie 1 - Version 1 - Version 1 - Versjon 1 • 03/06/2025

CE

www.pendulum.be

Snoezle[®]

Inhoudstafel *Table of Contents - Table des matières - Innholdsfortegnelse*

- 1** **Hallo, ik ben Snoezle** - P4
*Hi, I'm Snoezle
Bonjour, je suis Snoezle
Hei, jeg er Snoezle*
- 2** **Maak je Snoezle klaar** - P5
*Prepare your Snoezle
Préparer votre Snoezle
Forbered din Snoezle*
- 3** **Leer je Snoezle beter kennen** - P18
*Get to know your Snoezle better
Apprenez à mieux connaître votre Snoezle
Bli bedre kjent med Snoezle*
- 4** **Berg je Snoezle op** - P22
*Store your Snoezle
Stockez votre Snoezle
Oppbevar din Snoezle*
- 5** **De Snoezle thema's** - P26
*The Snoezle themes
Les thèmes du Snoezle
Snoezle-temaene*
- 6** **Wissel je Snoezle thema** - P27
*Switch easily between snoezle themes
Passez facilement d'un thème à l'autre
Bytt enkelt mellom snoezle-temae*
- 7** **Gezellige Snoezle Accessoires** - P30
*Cosy snoezle accessories
Accessoires snoezle douillets
Koselig snoezle-tilbehør*
- 8** **End of life - Recycle** - P31
*Fin de vie - Recyclage
Utrangering - resirkulering*
- 9** **Reiniging & wassen** - P33
*Cleaning & washing
Nettoyage et lavage
Rengjøring og vasking*
- 10** **CE conformiteit, veiligheid en garantie** - P35
*CE conformity, safety and warranty
Conformité CE, sécurité et garantie
CE-samsvar, sikkerhet og garanti*

1

Hallo, ik ben Snoezle®

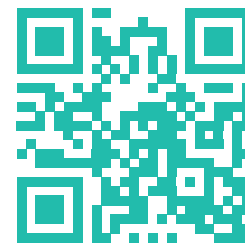
Hi, I'm Snoezle - Bonjour je suis Snoezle - Hei, jeg er Snoezle

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EN The Snoezle is a mobile multisensory relaxation product intended for indoor use by children and adults with or without care needs. Its purpose is to provide a calming or stimulating experience through light and sound stimulation. The product is not intended as a medical device.

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EN **The Snoezle tent:** The tent is made of recycled textile with a colourful print in a theme of your choice. The border features a multicoloured LED strip.

The new Snoezle module: All the electronics are in a separate module. It controls the LEDs and also contains a subwoofer. Together with two new speakers, it provides an immersive 8D immersive sound.

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EN The Snoezletent is made of:

- Recycled textile (100% polyester) with a colourful print in a theme of your choice.
- The edge is fitted with a multicoloured LED strip (60 LEDs per metre / IP67)
- Tent pole

Electronics and Snoezle control module:

- 2 speakers connected via audio cables to the module
- 1 subwoofer speaker integrated in the module
- 1 control panel with soft-touch operating buttons integrated in the module
- Built-in Li-ion Battery LP21700B 7.4V 4500mAh
- USB-C charger 5.0V 3.0A 15 Watt max.

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EN The Snoezle and module may be used and stored in indoor environments with:

- An ambient temperature between +10 °C and +40 °C
- A relative humidity between 30% and 75%

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- EN** 1. Remove the elastic to open the Snoezle.

ATTENTION! Make sure that you have enough space and that there is no-one in the way.

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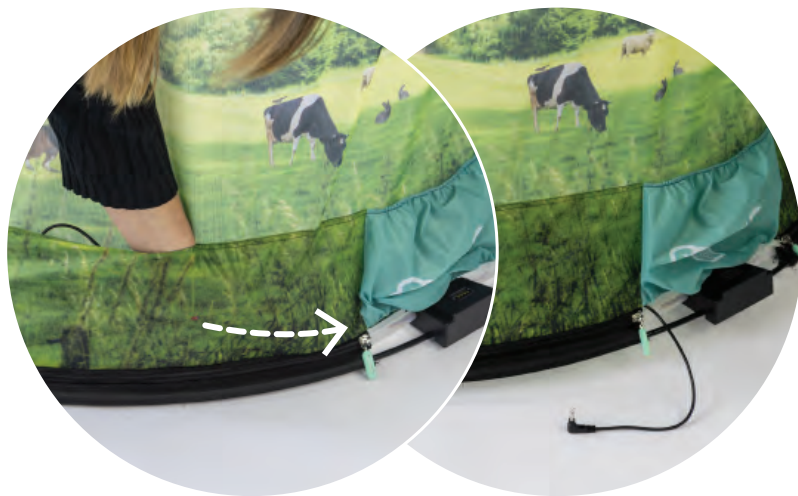
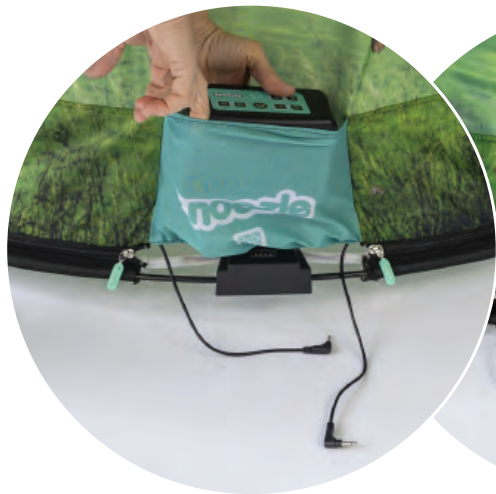
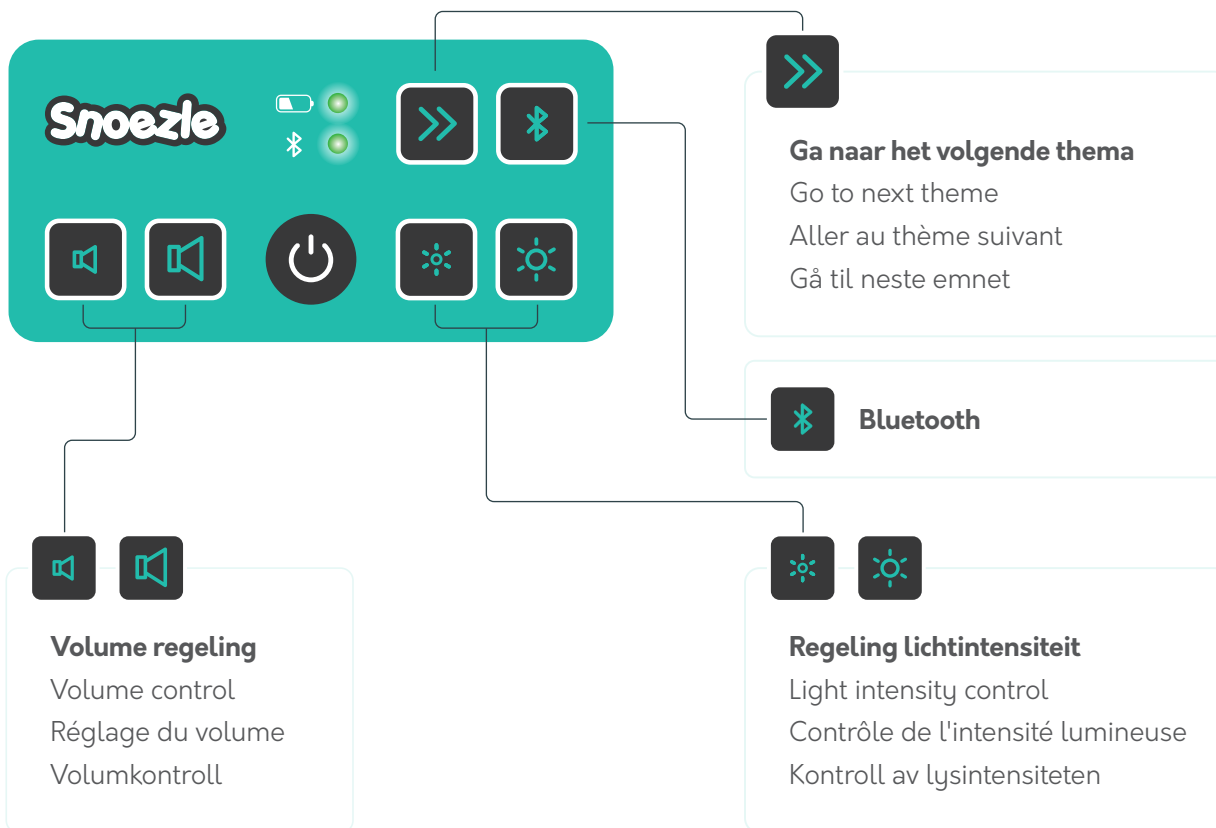
EN 5 Music Themes

The module contains 5 different Themes: Underwater, The Forest, By the Sea, Space & Nordic. Each theme exists in two versions: an intense and calm mode. Each press of the button takes you to the next theme.

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Nordisk

9. *Intens - Intense - Intense - Intensiv
10. **Rustig - Quiet - Calme - Stille

*The **intense mode** is a more stimulating mode. In this mode, there is a bit more activity regarding light and sound.

The **quiet mode is more relaxing, less stimulating.

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EN Bluetooth

Select Bluetooth and pair your own device (smartphone, other...) to play your own music. Find the Snoezle connection in your list of Bluetooth devices and pair your device with the Snoezle module ‘SNOEZL-XXXXXX’ The Alphanumeric combination of six digits and/or letters after ‘SNOEZL-’ is unique per module. See example of possible combination in box at bottom of page.

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Connect the USB-C charger for charging.

- The module can be easily charged both when mounted on the Snoezle and separately.

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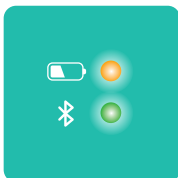
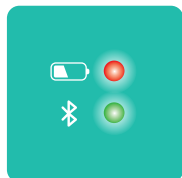
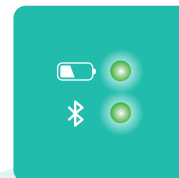
EN Thanks to this tripod, you can mount your Snoezle everywhere: above a chair or a wheelchair, a seat, a bed... The height is adjustable.

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EN Removing the module is not necessary. The tent is simply folded in two, super easy. This way you can move the Snoezle or put it aside.

Caution!

The size you fold* it in is larger than the one the Snoezle is sent in**.

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EN Has your Snoezle reached the end of his life? You can return your old Snoezle to us. We recycle it and give certain parts a second creative life.

Don't throw away electrical products with the regular garbage, but bring them in to a collection point for electrical waste. There is no charge for this service.

This symbol indicates that the product must not be disposed of as household waste and must be taken to an appropriate collection point for recycling. Proper disposal and recycling helps protect natural resources, human health and the environment. For more information on the disposal and recycling of this product, contact the municipality where you live, the waste disposal authority or the store where you purchased this product.

DO NOT attempt to remove the rechargeable lithium ion battery from this product.

Dispose of used batteries properly according to local regulations.

The rechargeable lithium ion battery in this product should only be removed by a qualified professional. For further information contact [pendulum](http://www.pendulum.care) or visit www.pendulum.care

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EN Tent and textiles

- Can be cleaned with clinell universal wipes
- After removing textiles (see chapter 6 change topics), the textiles can be cleaned with:
- Hand wash programme at max 30°C
 - Hang to dry

⚠ Note: Do not use bleach, do not tumble dry!

Speakers and module:

1. Remove the speaker from its cover
2. Clean the outside of the speaker with a soft, damp cloth (water only).

⚠ Note:

- Do not use solvents, chemicals or cleaning agents containing alcohol, ammonia or abrasives.
- Do not spill liquid into openings.
- Make sure the speaker is thoroughly dry before replacing the cover.

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EN Pendulum hereby declares that this product complies with the essential requirements and relevant provisions under applicable EU directives. **The full declaration of conformity can be found at www.pendulum.care/compliance.**

Important safety instructions

1. Read these instructions.
2. Keep these instructions.
3. Observe all warnings.
4. Follow all instructions.
5. Use only components or accessories specified by the manufacturer.
6. Refer all servicing to authorised personnel. Maintenance is required if the unit is damaged, for example if a power cord or plug is damaged, if the unit does not operate normally, or if it has been dropped

If the tent pole arch breaks, do not use the Snoezle again until the tent pole is replaced with an original Snoezle part.

Keep the Snoezle tent away from fire and heat sources.

Use only the included USB-C charger and check before use that no electrical wires have become disconnected or that the insulation of the cables is not damaged. If this is the case, unplug it immediately and replace the damaged part.

For your own safety, never try to repair the appliance yourself. This will void the warranty. Do not open the device, the device has no service parts.

In case of problems, contact the technical department at: info@pendulum.be

Warranty

This product comes with a statutory warranty of ****2 years**** from the date of purchase, in accordance with European Directive 1999/44/EC and national legislation. This warranty covers defects in materials and workmanship under normal use as described in this manual.

****The warranty does not apply to:****

- Damage caused by misuse, fall or impact damage
- Normal wear and tear (e.g. zips, fabrics)
- Damage caused by the use of non-original parts or chargers
- Attempts to repair or modify the product yourself

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