
# PROTAC®

INTEGRATING SENSES

## Protac® BallBase

- DA Brugsvejledning
- EN Operations manual
- DE Gebrauchsanweisung
- FI Käyttöohje
- FR Mode d'emploi
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- ES Instrucciones de uso
- IT Istruzioni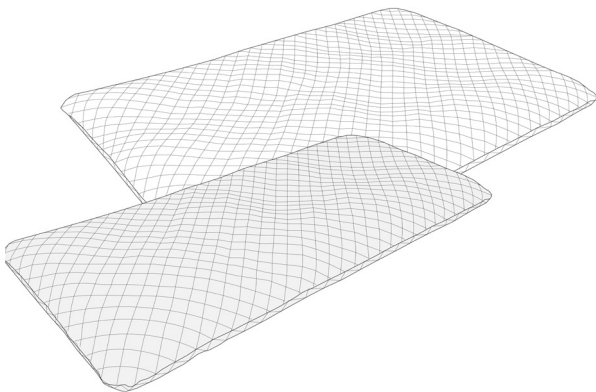

|                 |    |
|-----------------|----|
| Dansk .....     | 6  |
| English.....    | 10 |
| Deutsch .....   | 14 |
| Suomi .....     | 18 |
| Français .....  | 22 |
| Nederlands..... | 26 |
| Svenska .....   | 30 |
| Español .....   | 34 |
| Italiano.....   | 38 |

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# English

EN

|                                      |    |
|--------------------------------------|----|
| Marking .....                        | 11 |
| Incidents - adverse events .....     | 11 |
| Traceability .....                   | 11 |
| Key to symbols - product label ..... | 11 |
| Safety - general precautions .....   | 11 |
| Use of the product .....             | 12 |
| Use environment .....                | 12 |
| Indications .....                    | 12 |
| Contra-indications .....             | 12 |
| Maintenance .....                    | 13 |
| Cleaning .....                       | 13 |
| Key to symbols - care label .....    | 13 |
| Service life and warranty .....      | 13 |
| Disposal .....                       | 13 |

You have chosen a Protac quality product. Protac guarantees that this product is of original Danish design and developed in consultation with experts.

Since 1994, Protac has developed, manufactured and sold sensory-stimulating aids to the healthcare sector in Denmark.

Today, Protac products are sold worldwide.

All Protac products have been developed on the basis of the theories of sensory integration, and they are supported by extensive research, documentation and case studies.

The theory of sensory integration by A. Jean Ayres forms the basis of the development of all Protac products. The theory is based on how the brain processes sensory impression and how stimulation of the muscle-joint-sense and the deep tactile sense has a calming and organising effect on the sensory system.

All Protac products contain balls, and it is the movement of the balls that provide the dynamic deep-point pressure on the body and stimulate the receptors in the deep tactile sense and the muscle-joint sense. This enhances the body awareness, alleviates sensory disorder and has a calming effect on both body and mind.

Further information can be found at [www.protac.dk](http://www.protac.dk).

## Marking

The products are CE-marked in accordance with Regulation (EU) 2017/745 on medical devices.

EU declaration of conformity can be found at [www.protac.dk](http://www.protac.dk)

## Incidents - adverse events

- Incidents related to the use of Protac® BallBase must be reported to Protac.
- Adverse events related to the use of Protac® BallBase must be reported to the National Competent Authority and to Protac.

## Traceability

Production month, year and batch number can be found on the product label.

Do not remove the product label from the product.

EN

## Key to symbols - product label

|                                                                                   |                                                                          |                                                                                   |                       |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------|
|  | Instructions for use                                                     |  | Maximum user weight   |
|  | Medical device                                                           |  | Article number        |
|  | Lot number                                                               |  | For indoor use        |
|  | Manufacturer                                                             |  | Production month/year |
|  | CE marked in accordance with Regulation (EU) 2017/745 on medical devices |                                                                                   |                       |
|  | Warning: See under Safety                                                |                                                                                   |                       |

## Safety - general precautions

|                                                                                     |                                                                                      |
|-------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
|    | Not for unsupervised use by users unable to care for themselves                      |
|    | Incontinence covers can cause strangulation and must be used with caution.           |
|   | Avoid smoking next to the product. Dropped embers or cigarettes can cause burns.     |
|  | Never leave switched on electrical appliances that emit heat on or near the product. |
|  | Do not use near open fire.                                                           |
|  | Do not use other balls in the product than those supplied by the manufacturer.       |

## Use of the product

Protac® BallBase is supplied ready for use. The mattress pad has a soft and an extra soft side to accommodate different needs for sensory stimulation. The mattress pad is to be used with a normal bedsheet.

The mattress pad is supplied with a nylon cover to protect the material against dirt. The protective cover can be kept on during first use.

Protac® BallBase can be used as a regular mattress pad for sleeping during the night.

Protac® BallBase can also be used during the day, if there is a need to calm down.

## Use environment

Protac® BallBase is for indoor use. The mattress pad can be used both in institutions and private homes.

## Indications

Protac® BallBase is a sensory-stimulating aid intended to compensate for a permanent disability.

Protac® BallBase contains balls which move and apply dynamic pressure on to the body, thereby stimulating the muscle-joint sense and the tactile sense. This increases body awareness and calms the brain and body, and the product thereby alleviates physical and mental unrest as well as sleeping difficulties.

## Contra-indications

Persons with pressure ulcers or persons at risk of developing pressure ulcers are not to use the product.

Pay attention to the user's reaction, if the user can't express his/her own condition.

## Maintenance

It is recommended to air the mattress pad on a regular basis.

## Cleaning

Protac® BallBase can be washed in a regular household washing machine. Cover and ball bags to be washed separately. The frequency of washing depends on use. When washing, please follow the instructions on the care label. Washing instructions can also be found on pages 42-43 of this manual.

Protac does not recommend the use of stain remover.

## Key to symbols - care label

EN

|                                                                                   |                             |                                                                                   |                       |
|-----------------------------------------------------------------------------------|-----------------------------|-----------------------------------------------------------------------------------|-----------------------|
|  | Maximum washing temperature |  | Line dry in the shade |
|  | Do not wash                 |  | Do not iron           |
|  | Can be tumble dried         |  | Do not dryclean       |
|  | Do not tumble dry           |  | Do not bleach         |

## Service life and warranty

The expected service life of the product is 5 years. This is based on correct use, cleaning, washing and maintenance.

Products used beyond the expected service life must regularly undergo a maintenance inspection which implies checking for wear and tear in fabric and seams.

The product comes with a 2-year warranty, see page 2.

The warranty will be invalid in case of:

- overload or excessive use of the product
- lack of or incorrect maintenance of the product

## Disposal

The product can be broken down into the following components for disposal:

- The plastic balls can be disposed of as hard plastic.
- The fabric can be disposed of as textiles.

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Protac A/S