Vedrørende miljøbeskyttelse:

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UK:

Koko-Nora® the small rhino Morriz is a heavy blanket for children, who may have difficulty in falling asleep or just need a time out during the day. Let the body of Morriz lay on top of the child's stomach/chest and let it give a light pressure to the body. What happens in a body when it is exposed to stimulation, as pressure, is that it releases Oxytozin, which is de-stressing and gives a calm feeling to a stressed body. Morriz have ears and horn which the child can use as a tactile stimulation, and beside this the surface is soft, which also gives tactile stimulation to restless fingers. Who knows maybe Morriz can give less sleepless nights.... Just pay attention to one thing, the child have to be able to remove it by themselves, in case they want to sleep without.

The Koko-Nora® product is designed with love, to give calmness and safety, for children with an unrestfull body from the age +1 year.

Size: H-30mm W- 470mm L-470mm weight: 1,5 kg

Composition:	100%	Polyester
Composition inside:	98%	Polyethylene
	2%	Polyester

Koko-Nora® only uses qualities that do not contain carcinogens. Therefore, no flame retardants have been used on the product.

Other practical informations:

Care instructions are at the back of the product. Koko-Nora® recommend that superficial dirt will be removed with a damp cloth and soap, since repeatedly wash will wear on the product and the warranty does not cover this. To protect the material, only use the product indoors. Keep the product away from sharp and hot objects as well as open flames. Koko-Nora® would like to guarantee that our products can withstand all kinds of bites, but we cannot. Teeth are counted as very sharp objects. If the person bites in the product, the product may leak, and therefore you should always inspect the product before use. It is recommended to use both hands when moving the product. Do not pull the product in any way. If you are transporting the product in a vehicle, the product must be secured to prevent it from flying around and injure anyone. If there is a hole in the product, the product must be removed from the person immediately. The Koko-Nora® product should be checked regularly for damage, as loose objects and its contents can be dangerous for children less than +3 years of age as well as people with dementia. Do not put the product in the microwave or another heat source.

Regarding environmental protection:

Help the environment by not disposing your product in the household waste. Cut the product into pieces and distribute the individual components such as plastic and fabric and according to general local regulations – in the relevant recycling systems.

Warnings:

If the person is severely restricted by motoric and/or physical conditions, the product should be used with special caution. Do not use the product if the person cannot remove it by themselves! For safety reasons, we generally recommend that people using an Koko-Nora® product should be under supervision during use. After the use of the product, the persons who require supervision must be kept out of reach of the product. Keep the airway clear when using the product. If the person has difficulty breathing, do not place the product on the chest, as the weight can have a negative effect on breathing and cause a choking hazard. All products with weight should be used with caution, as weight i.a. can cause blockage of blood circulation. An Koko-Nora® product should not be used near water as it may cause drowning.

Warranty:

In the event of manufacturing defects, Koko-Nora® provides a two-year warranty. This warranty is only valid if the product is used as directed. Damage caused by incorrect or improper use excludes the use of the warranty. The product can be transported in a bag. When used correctly, you get a product that lasts a long time.

For further questions, please feel free to contact Majbritt Hillerup. Keep this original instruction manual for future reference.

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Oberstoff:	100%	Polyester
Futter:	98%	Polyethylen
	2%	Polyester

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