# HOW TO USE LONE

When the child has to go up or down from the Lone chair, then place the arms to the side.

The weight of the arms may cause the child to use only one arm, so place the other arm next to the child. If the child is okay with it, both arms can be used. Start with 5-10 minutes and increase the time as needed.

When the Lone chair is not in use the arms must lay like an X on top of the stomach.

Watch the video on the website on how to use the chair!

## Washing:

## Warranty:

In the event of a manufacturing fault, Oliz Koko-Nora® provides a two-year warranty. This warranty is only valid if the product is used as intended. Damage caused by incorrect or inappropriate use is excluded from the warranty.

For further questions about the use of the Oliz Koko-Nora® product, please contact the person responsible for the product, Majbritt Hillerup.

Order:

Year:

Product: Lone

EN21856:2022, tested according to the standard EN71-1-2-3

# LONE

Oliz  
Koko-Nora®  
Danish design

— the resting chair —

... is a specially designed chair for people who often experience physical tension and restlessness. Provides you with relaxation and relief in everyday life. Lone gives peace to the body and the feeling of being embraced, as the heavy arms lie across the body of the person in need.

UK

Danish design- produced in EU - [www.oliz.dk](http://www.oliz.dk)  
Oliz Koko-Nora- Orionvej 2A- 7430 Ikast -DK -tlf +45 20 78 79 54

# HOW TO TREAT LONE

Do not jump or pull the product

to avoid injury on the chair!

Keep the chair away from

teeth, fire and sharp/hot objects!

Clean the product with


water and soap by using a wet soft cloth.

For indoors

use only!

Stay on top of the product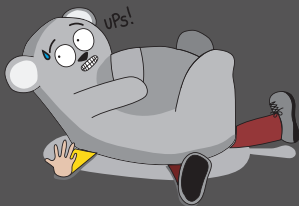

to avoid the risk of suffocation!

Fill the chair in a closed room

and clean the room afterwards!

## Other practical information:

Maximum weight that can be put on the chair is 90 kg, and should be used only by people above the age of 3 years. Two people are needed to move the chair. Do not move the chair yourself! If transporting the product in a vehicle, the product must be secured to prevent it from moving and injuring someone. Check regularly for any damage or loose objects and contents, which can be dangerous for children under 3 years of age. Do not throw the product, the weight of the product can cause damage to both the product and to people in its vicinity. Make sure the floor under the chair is clean, sand and other sharp objects will damage the fabric. This damage will not be covered by the warranty. It is important to keep an eye on the person during use as the product is heavy and this can physically affect the body. Should be only used with the person's consent. If the person feels discomfort while using the product and refuses the product in general the product must be removed and not used for that person. When starting to use the product, it is a good idea to first try it for 5-10 minutes to test its effect, then increase the time interval according to one's needs. If the person gets too used to the product, the effect will wear off. Once the desired effect has been achieved, the product must be removed carefully. If a hole is present get out of the chair immediately, and place the chair in a closed room, and follow the information to the environment. To maintain the best shape and use for the seat then fill it up when it looks flat. For info about filling of EPS contact Majbritt Hillerup at [mh@oliz.dk](mailto:mh@oliz.dk)

Always read and keep this original manual for future reference.

## Dimentions:

Lenght: 1250 mm

Width: 870 mm

Height: 850 mm

Weight: 10 kg

Each arm: 2kg

Composition: 100% imitated leather/PU material

Inside composition: 8% Polyethylene, 2% polyester, 90% EPS pearls

Oliz Koko-Nora® only uses qualities that do not contain carcinogenic substances. Therefore, no flame retardants have been used on the product.

## Warnings:

If the person is severely limited by motor and/or physical conditions, the product must be used with particular caution. Do not use the product if the person is unable to remove it themselves! For safety reasons, we generally recommend that people using an Oliz/Koko-Nora® product are supervised during use. After using the product, persons who need supervision must be kept out of reach of the product. Ensure that the airways are kept clear while using the product. If the person has breathing problems, the product must not be placed on the chest, as the weight can have a negative effect on breathing and cause a suffocation hazard. All products with weight should be used with caution, as the weight can cause blockage of blood circulation. An Oliz/Koko-Nora® product must not be used near water, as it can cause drowning. If the person has heart or lung disease, you must contact your doctor if you want to use this product

## Enviroment:

Protect the enviroment, do not throw the chair in the household waste. Separate the product into pieces and distribute the individual components such as plastic, EPS stuffing and fabric according to general local regulations- in the appropriate recycling systems

# HOW TO FILL LONE

1. Flip the chair, so you can reach the bottom


2. Open and close the zipper with paper clip, by using the clips in the zip head.

3. Next open the hook and loop fastener inside

4. Pull the hook and loop fastener outside of the chair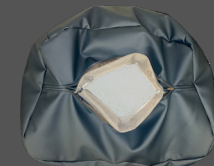

5. Flip or roll up the hook and loop fastener, like a sleeve for easier access

6. Fill up the seat with EPS filling and carefully close it.

Remember to remove the paper clip after use!

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